


Improve Patient Care, Reduce Staff and Chair Time

Hemostatic HemCon® Strip for Dialysis

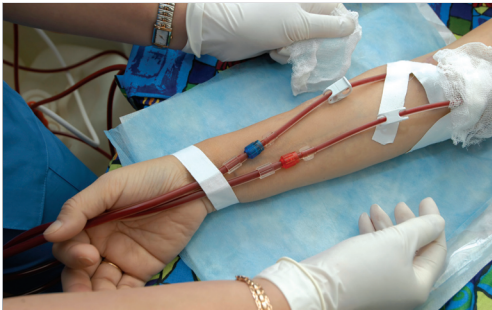

Effective bleeding control for access procedures

- AV Fistula and Graft bleeding upon sheath removal
- AV Fistula and Graft bleeding post de-clot procedures
- Catheter bleeding
- Bleeding from puncture access sites post hemodialysis



What the experts say...


"We use the HemCon Strips on de-clot procedures that do require sutures as the patient may still be bleeding due to heparin or TPA that was used. They work great and we haven't had any complaints from our Dialysis Centers here on removing them with water on their next treatment."

—Amy Howser, R.T. - RMS Lifeline, El Paso, TX

Advantages of HemCon Strips

- Shortens manual compression time
- Helps preserve fistulas and grafts
- Frees up staff and physician time
- Prevents suturing in some patients
- Effective on anticoagulated patients
- Offers an antibacterial barrier that protects the access site and may help to reduce the risk of infection
- No delay in dialysis preparation time

INSTRUCTIONS FOR USE

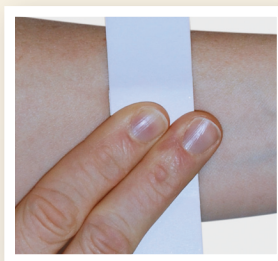

1. Remove sheath as per your access center/hospital protocol.
2. Allow a small amount of blood, approximately the size of a nickel, to surround the puncture site. **Blood is required for the bandage to seal the wound.**
3. Without removing the liner from the adhesive wings, fold the wings up and place the white, center pad in contact with the source of bleeding.
4. Hold pressure for 3-5 minutes or until bleeding is controlled.



5. Once bleeding is controlled, release liner from adhesive wings and secure HemCon Strip on patient's arm.
6. The HemCon Strip can be left in place for up to 48 hours. Keep bandage dry.

Removal Instructions: Patient can remove the HemCon Strip by soaking it with water during preparation for the next scheduled dialysis session.



For more information please contact:
HemCon Medical Technologies, Inc.
10575 SW Cascade Avenue, Suite 130,
Portland, OR 97223-4363
U.S. & Canada Toll Free: 877.247.0196
Phone: +1.503.245.0459
www.hemcon.com • info@hemcon.com

HemCon Medical Technologies, Inc. created and manufactures the HemCon Bandage, a revolutionary way for medical professionals to treat and control bleeding. First used in 2003 by the U.S. Army to reduce the number of battlefield deaths due to bleeding, HemCon Bandages remain a top choice for Armed Forces and civilian medical professionals worldwide to achieve hemostasis and improve patient care.